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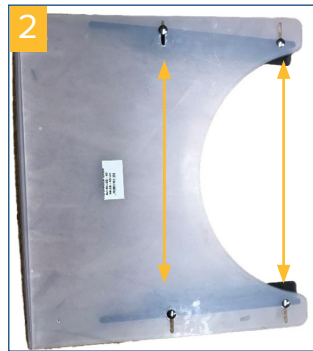
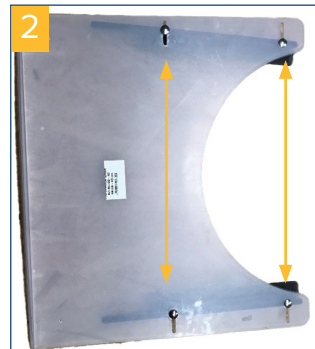
Lieferumfang: Top, 4 rails, 4 height adapter, 4 screws, 4 washers, 4 nuts


1. Loosening width adjustment

To be able to adjust the width of the Wheelchair tray, the rails must first be loosened, picture 1. To do this, unscrew the nuts so that the rails can be moved along the drill slots, picture 2.

2. Width adjustment

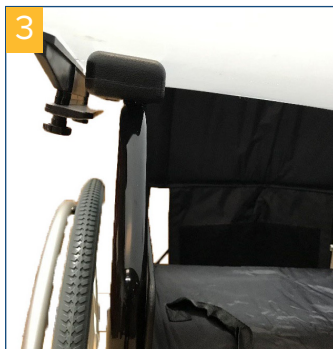Now place the wheelchair tray on the armrests of the wheelchair so that both rails (all four screws) are positioned outside. This means that the table is initially set „too wide“, picture 3. push the rails towards the armrests as far as they will go. Now you can tighten the handle nuts on the underside, picture 4.


3. height adjustment

If the wheelchair tray can still be moved up and down after it has been fixed at the sides, the height of the fixation can also be adjusted. please add or remove the hight adapter. ti djsut the hight of the wheelchair tray. Picture 5

4. Removing / adding the height adapter

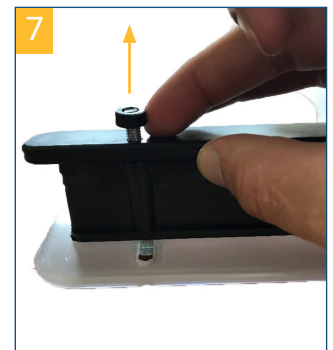
loosen and release the nut, picture 6. Remove the washer, picture 7.

Remove the rail, picture 8. Finally, the height adapters can be removed or attached as required to ensure that the wheelchair tray is fixed flush. The height fixation must be reattached, as this is necessary for tensioning the table.

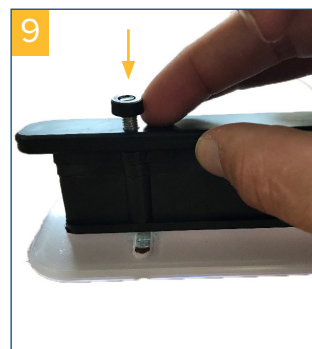Then put the washers back onto the screw, picture 9. add the nuts back onto the screw picture 10.

Notes about: safety, storage, maintenance, care, cleaning, repair, disposal and use refer to the instructions from the manufacturer TheraMedical GmbH.