USER MANUAL

VICAIR®

LIBERTY BACK

relief by air

EN	USER MANUAL	3
NL	GEBRUIKERSHANDLEIDING	8
DE	GEBRAUCHSANWEISUNG	14
FR	MANUEL DE L'UTILISATEUR	21
DA	BRUGERVEJLEDNING	27
SV	BRUKSANVISNING	32
NO	BRUKERMANUAL	38
FI	KÄYTTÖOHJE	43
ES	MANUAL DE USUARIO	49
PT	MANUAL DE UTILIZADO	55
IT	MANUALE UTENTE	61
GR	ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ	67
HR	KORISNIČKI PRIRUČNIK	73
SL	UPORABNIŠKI PRIROČNIK	79
PL	INSTRUKCJA UŻYTKOWNIKA	85
ZH	用户手册	92
JP	ユーザーマニュアル	96

Dear customer,

Thank you for choosing a Vicair® product. We are sure you will appreciate the simplicity, reliability and fantastic effect. For quick installation you can use the Quick Installation Guide provided with your cushion. In addition, please read and understand this user manual before use.

For more information or to (re)order manuals, contact your country's distributor.
Go to vicair.com for distributor information.

INTRODUCTION

The Vicair Liberty back is a wheelchair back support cushion for users who require comfort in combination with a low to medium level of support and pressure injury prevention. It is designed for use in combination with the Vicair Strap back.


INDICATIONS

It is used to provide comfort, a low to medium level of stable body support and to help prevent pressure injuries.

CONTRAINDICATIONS

When used as intended, there are no known contraindications.

MAIN COMPONENTS

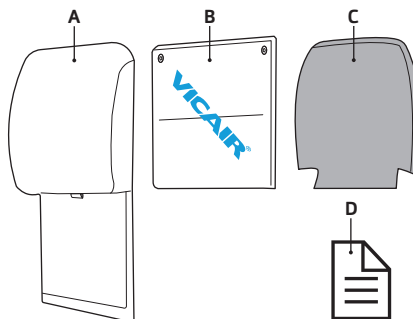- A.** Cushion cover (Comfair Cover)
- B.** Cushion
- C.** Foam insert
- D.** Quick Installation Guide

GENERAL PRODUCT SPECIFICATIONS

Product name:	Vicair Liberty back
Manufactured by:	Vicair B.V., The Netherlands
Warranty:	24 months
Weight (excl. cover):	See information in the back of the manual
Available sizes:	See information in the back of the manual
Used materials:	All latex free
Cushion:	Polyvinylchloride
Filling:	Air-filled SmartCells™ and silicon oil
Foam insert:	Closed-cell polyethylene foam
Comfair Cover:	Front: knitted polyester fabric. Sides: knitted polyester 3D fabric. Back: nylon canvas fabric. Machine washable. Cover complies with EN1021-1, 1021-2.

PRECAUTIONS AND SAFETY INSTRUCTIONS

A Vicair wheelchair back support cushion may only be used as a wheelchair back support cushion. The use of this product in any other way, is considered to be misuse. Vicair B.V. cannot be held responsible for damage caused by misuse, improper use or negligence. Make sure that you read and understand this manual completely.

Warning! Vicair B.V. recommends to install the Vicair Liberty back under direct supervision of a therapist or Assistive Technology Professional.

Warning! Your back and spine should be supported by the air filled SmartCells within the cushion and should not touch the foam insert. Checking for “bottoming out” should be done during first use and after that on a regular basis (at least once a month).

Warning! Do not use as a shower or bath cushion.

Warning! Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, consult your therapist or Assistive Technology Professional.

Warning! In case of soiling with body fluids, and when transferring the product to another user, the product must be disinfected to avoid cross contamination.

Warning! Every three months, check the product for wear and deflated cells. In case of serious damage or a large number of deflated cells, do not continue to use the product. Consult your Assistive Technology Professional.

Warning! Do not immerse this cushion in water or any other liquid and do not wash in a washing machine.

Warning! Using the Vicair Liberty back may affect the static and dynamic behaviour of the wheelchair. Check and test the balance and stability of the wheelchair with the aid of your therapist or Assistive Technology Professional.

Caution! The Vicair Liberty back is used in combination with the Vicair Strap back. Consult the Vicair Strap back user manual for information about installation, use, maintenance and cleaning of this product.

Caution! Do not use or leave the product close to a naked flame or a high temperature heat source.

Caution! Do not open the valve. Opening might cause silicon oil leakage.

Caution! Do not inflate the cushion.

Caution! Do not dry clean, steam clean or autoclave the product. The product cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the product and/or the product's cover will age more rapidly.

Caution! Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.

Caution! Excessive washing of the cushion cover may influence the aging of the cover.

INSTALLATION

Warning! Vicair B.V. recommends to install the Vicair Liberty back under direct supervision of a therapist or Assistive Technology Professional.

Warning! Using the Vicair Liberty back may affect the static and dynamic behaviour of the wheelchair. Check and test the balance and stability of the wheelchair with the aid of your therapist or Assistive Technology Professional.

Caution! The Vicair Liberty back is used in combination with the Vicair Strap back. Consult the Vicair Strap back user manual for information about installation, use, maintenance and cleaning of this product.

Caution! Do not open the valve. Opening might cause silicon oil leakage.

Caution! Do not inflate the cushion.

Notice! Always use cushion with cover.

Perform the following steps when installing the Vicair Liberty back

These steps correspond to the installation images 1 – 4 on the Quick Installation Guide:

- Check cushion size. The distance between the back posts of the wheelchair (X) determines the cushion size. X (min-max):
 $36-40\text{ cm}/14\frac{1}{2}-16'' = \text{LBC}/1$
 $40-44\text{ cm}/16-17\frac{1}{2}'' = \text{LBC}/2$
 $44-48\text{ cm}/17\frac{1}{2}-19'' = \text{LBC}/3$
- Check if the size of the cover matches the size of the cushion.
- Open the zipper on the topside of the cushion cover and check if cover contains the foam insert. The round side of the mushroom-shaped foam insert should be facing up.
- Insert cushion in cover. The print at the back side of the cushion indicates its back/front and upside. The side of the cushion cover with loop strips and a vanity flap attached to its outside is the back side of the cover. The foam insert should be between the back side of the cushion and the back side of the cushion cover.
- Pull the loop strips attached to the inside of the cushion cover over the top of the foam insert and attach it to the hook strip attached to the top-back side of the cushion.
- Close zipper. Make sure to not damage the cushion and/or foam insert.

If not yet installed: install your Vicair Strap back. Consult the Vicair Strap back user manual for the installation instructions of this product.

Install the Vicair Liberty back on your Vicair Strap back

These steps correspond to the installation images 5 – 9 on the Quick Installation Guide:

- Lift the vanity flap and place the cushion against the Vicair Strap back. The loop strips on the cushion cover should hook onto to the hook strips on the Vicair Strap back.

- Sit in the wheelchair for a few minutes, to allow the back support cushion to adapt to your body shape.
- Your therapist or Assistive Technology Professional can now adjust the straps of the Vicair Strap back to the contours of your back.
- Fold the cushion's vanity flap over the Vicair Strap back.
- Fold the end of the vanity flap under your seat cushion. Optional: use the loop strips at the end of the vanity flap to attach it to the wheelchair seat.

Checking for "bottoming out"

Image 10 - Quick Installation Guide

Warning! Your back and spine should be supported by the air filled SmartCells within the cushion and should not touch the foam insert. Checking for "bottoming out" should be done during first use and after that on a regular basis (at least once a month).

- Check for bottoming out, after sitting with your back against the back support cushion for approximately 5 minutes.
- Lean slightly forward, let your therapist or Assistive Technology Professional slide a hand between your back and the cushion, then lean back again.
- Ask your therapist or Assistive Technology Professional to check if your back is supported by at least 2,5 cm / 1" of SmartCells.
- Do not use the cushion if bottoming out occurs. In most cases, using a different cushion model or cushion size will solve the problem.

Stability

Warning! Using the Vicair Liberty back may affect the static and dynamic behaviour of the wheelchair. Check and test the balance and stability of the wheelchair with the aid of your therapist or Assistive Technology Professional.

USE

Warning! Do not use as a shower or bath cushion.

Caution! The Vicair Liberty back is used in combination with the Vicair Strap back. Consult the Vicair Strap back user manual for information about installation, use, maintenance and cleaning of this product.

Caution! Do not open the valve. Opening might cause silicon oil leakage.

Caution! Do not inflate the cushion.

Caution! Do not use or leave the product close to a naked flame or a high temperature heat source.

Checking for redness of the skin

Image 11 - Quick Installation Guide

Warning! Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, consult your therapist or Assistive Technology Professional.

Ready for use

This Vicair product is ready for use.

Conditions for use and storage

- **User weight:**
Minimum: 15 kg / 33 lbs, maximum: 150 kg / 330 lbs
- **Temperature:**
Minimum: -10 °C / 14 °F, maximum: 50 °C / 122 °F
- **Use at high altitude**
Notice! Vicair B.V. recommends not to use the cushion above 2400 m (8000 ft) above sea level or below 50 m (164 ft) below sea level. Because each SmartCell is filled with air, the cushion may behave differently when used at high altitude or on an airplane.

- **Air humidity:**

Minimum: 0%, maximum 100%

Use of the Vicair back support cushions in motor vehicles

Whenever possible, transfer out of the wheelchair and into a car manufacturer-installed vehicle seat without the cushion, and use the vehicle's crash-tested occupant restraint system.

As an alternative for your Vicair wheelchair cushion, the Vicair Allrounder O2 (which can be fastened to the user's body) can be used if you require additional protection in a manufacturer-installed vehicle seat during transportation. When the wheelchair is used as the vehicle seat, the applicable rules and regulations with regard to securing the wheelchair and the wheelchair user during transport should be followed.

There is no data available about the influence on safety of a Vicair wheelchair cushion used in a crash-tested wheelchair.

Re-use - Transferring the product to another user

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. The product has to be thoroughly cleaned and disinfected, before another person is allowed to use it. See chapter 'Maintenance and cleaning' for instructions.

Expected lifetime

The expected lifetime of this product is two years when used in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective lifetime can vary according to frequency and intensity of use.

Replacing the cover

Under normal use, Vicair B.V. recommends to replace the cover every year. Replacement covers can be purchased separately.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulation. Ensure the product is cleaned prior to disposal to avoid any risk of contamination.

MAINTENANCE AND CLEANING

Warning! Every three months, check the product for wear and deflated cells. In case of serious damage or a large number of deflated cells, do not continue to use the product. Consult your Assistive Technology Professional.

Warning! Do not immerse this cushion in water or any other liquid and do not wash in a washing machine.

Caution! The Vicair Liberty back is used in combination with the Vicair Strap back. Consult the Vicair Strap back user manual for information about installation, use, maintenance and cleaning of this product.

Caution! Do not open the valve. Opening might cause silicon oil leakage.

Caution! Do not inflate the cushion.

Caution! Do not dry clean, steam clean or autoclave the product. The product cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the product and/or the product's cover will age more rapidly.

Caution! Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.

Caution! Excessive washing of the cushion cover may influence the aging of the cover.

Cleaning the Vicair Liberty back

Cleaning the cushion cover

Remove the cushion and foam insert before cleaning. The cushion cover can be machine washed inside-out. Follow the washing instructions on the label attached to the inside of the cover. (60 °C = 140 °F)

Cleaning the cushion

Wipe the cushion with a damp cloth using a mild detergent. Do not machine wash the cushion and do not immerse in water or any other liquid.

Cleaning the foam insert

Clean the foam insert by using a disinfectant spray.

Disinfecting the Vicair Liberty back

Warning! In case of soiling with body fluids, and when transferring the product to another user, the product must be disinfected to avoid cross contamination. Clean the cushion, cover and foam insert as instructed above. Preferably use an antibacterial detergent. After cleaning spray the cushion and cover with a disinfectant.

REPORTING OF SERIOUS INCIDENTS

Serious incidents from use of Vicair products should be reported to the manufacturer and the relevant competent authority.

WARRANTY

Please see www.vicair.com for complete information on warranty processes, terms and conditions.

Vicair B.V. offers a limited warranty against defects in workmanship and materials, provided the product has been used normally. Any damage arising from abnormal use, or caused by improper handling, cleaning, or negligence, is excluded from this warranty.

Warranty period

From original date of purchase: 24 months.

Should a defect in materials or workmanship occur within the above mentioned warranty period, Vicair B.V. will – at its own choice – repair or replace the product free of charge. Remedies for breach of express warranties herein are limited to repair or replacement of the product.

In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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ProPharma Group MIS Limited