Video Guides

Get started! **2**

English **7**

Deutsch **10**

Français **13**

Nederlands **16**

Dansk **19**

Swedish **22**

Español **25**

Carbon MRI Rollator


by  **ACRE**

**This rollator is MRI-compatible up to 3 Tesla/
This rollator is MRI-compatible up to 3 Tesla/
Dieser Rollator ist bis zu 3 Tesla MRT-kompatibel/
Déambulateur compatible IRM jusqu'à 3 teslas/ Deze
rollator is MRI-kompatibel tot 3 tesla/ Denne rollator
er MR-kompatibel op til 3 tesla/ Denna rollator är
MRI-kompatibel upp till 3 tesla/ Este andador es
compatible con resonancias magnéticas de hasta
3 teslas**


GET STARTED!

**Unfolding/ Auseinanderfalten/ Ouverture du
rollator/ Uitvouwen/ Udfoldning/ Fälla upp rol-
latorn/ Despliegue**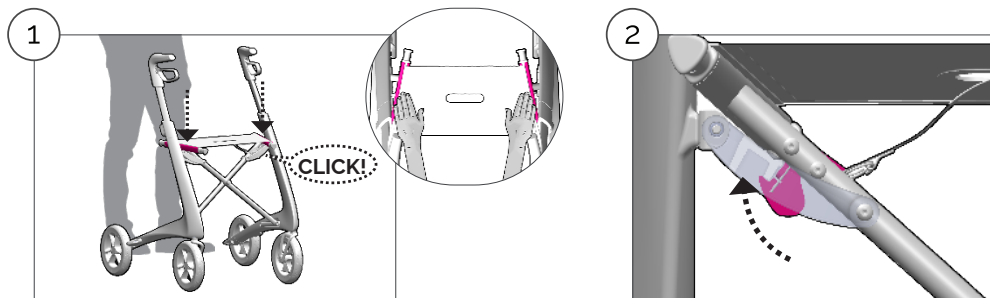


- ⚠ **EN** Push the seat rails down until you hear a click. Check that the hook is locked in place.
- ⚠ **DE** Den Sitz nach unten drücken bis ein hörbarer Klick erfolgt. Überprüfen Sie ob der Verriegelungshaken richtig eingerastet ist.
- ⚠ **NL** Duw naar Beneden aan beide uiteinden van het zitje totdat u én klik hoort. Gelieve na te kijken dat het haakje onder het zitje goed is vastgeklikt.
- ⚠ **FR** Poussez vers le bas de l'assise jusqu'à ce que vous entendiez un click. Veuillez vérifier que le crochet d'accrochage en dessous du siège soit bien fixé.
- ⚠ **DA** Pres ned på sædets yderramme, indtil du hører et tydeligt "klik". Tjek, at låsekrogen under sædet er i korrekt indgreb.
- ⚠ **SV** För ned sitsskenorna tills du hör ett klick. Kontrollera att kroken är i låst läge.
- ⚠ **ES** Empuje las guías del asiento hacia abajo hasta que oiga un clic. Compruebe que el gancho está bloqueado.

Folding/ Zusammenfallen/ Pliage du rollator/ Dichtklappen/ Foldning/ Fälla ihop rollatorn/ Pliegue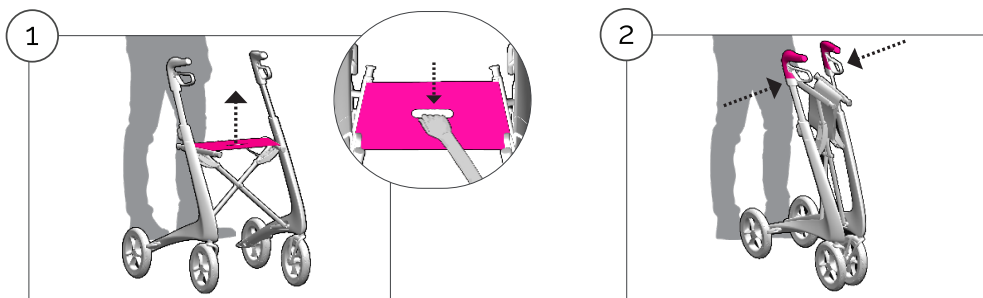

- EN** Lift the seat by the cut-out in the center and push the handles toward each other.
- DE** Heben Sie die Sitzfläche, und schieben Sie die Griffe zueinander.
- NL** Plaats uw hand door de gleuf in het zitje en trek het zitje naar omhoog. Kunt u best beide handgrepen naar binnen duwen.
- FR** Soulevez le siège par la fente au centre de l'assise. Poussez les deux poignées l'une contre l'autre.
- DA** Tag fat i sædet ved udskæringen, træk opad og skub håndtagene mod hinanden.
- SV** Lyft upp sitsen i utskärningen i mitten och för handtagen mot varandra.
- ES** Levante el asiento por la ranura central y empuje las empuñaduras una hacia la otra.

Height adjustment/ Einstellung der Griffhöhe/ Réglage en hauteur des poignées/ Hoogte instellen van de handgrepen/ Højdejustering af håndtag/ Höjdjustering/ Ajuste de altura

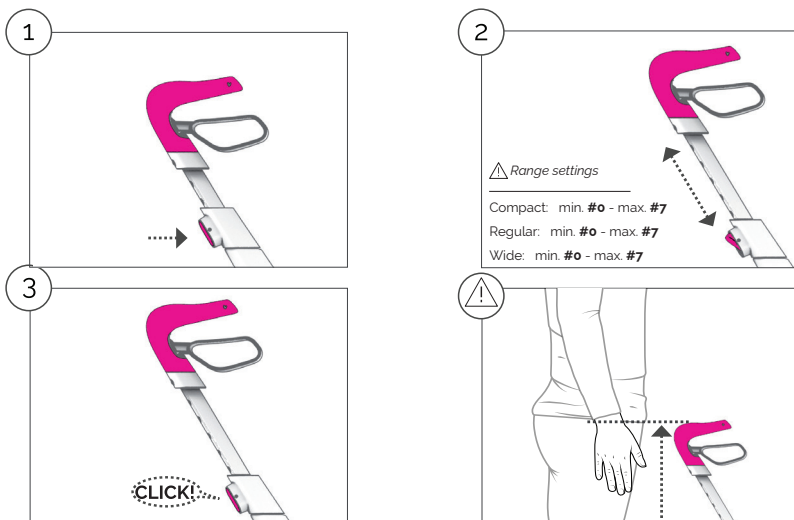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
English

Congratulations with your new Carbon MRI rollator. In order to utilize all the possibilities the Carbon MRI provides, it is important that you carefully study this user manual.

Technical specifications and overview of accessories are available toward the end of the user manual.

Brakes

Carbon MRI is equipped with two independent brakes acting on the rear wheels.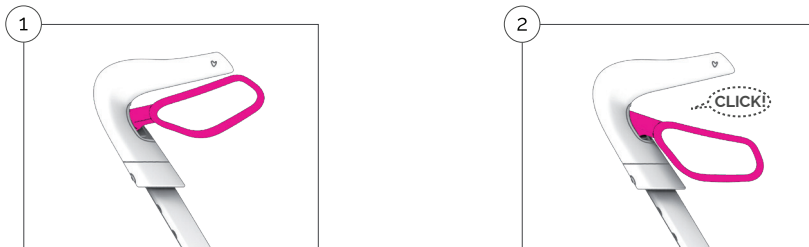


1. Pull the brake levers upwards to activate the brake. The lever moves back to the neutral position when the brake lever is released.
2. The parking brake is activated by pressing the brake lever down until you hear a click sound. Release the parking brake by pulling the brake lever back to the neutral position.

- ⚠ Always engage both parking brakes when you sit on the seat.
- ⚠ Under normal use both brakes should be activated simultaneously.
- ⚠ Be aware that when only one brake is activated, the rollator will turn to the same side as the engaged brake.
- ⚠ Do not store with the parking brake activated as prolonged activation of the parking brake can leave visible marks on the wheel and decrease the braking performance.

Adjusting the brakes


The brake should be adjusted if the brake force applied to the wheel is insufficient (this can occur if the tires are worn) or the parking brake is difficult to activate (which is most often seen after replacement of a worn down tire). Always set the handle to the desired height before the brake is adjusted as the position may cause a little variation of the brake adjustment.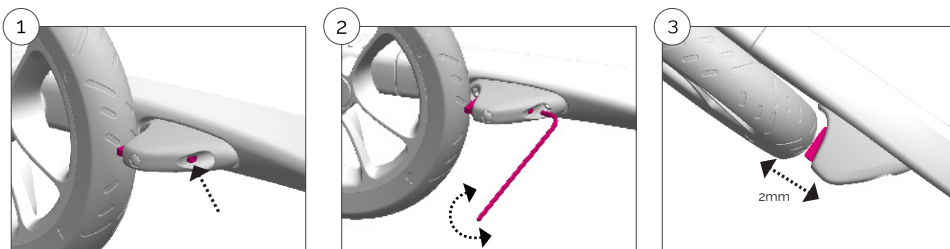

1. Adjust the brake with a 3 mm Allen wrench. Insert the Allen wrench into the adjustment screw through the hole in the front of the brake unit.
2. The brake is tightened by turning the adjustment screw clockwise (viewed from the front) and loosened by turning it counter-clockwise (viewed from the front).
3. Check the distance between the brake and the wheel. It should be 2mm.

Curb lifter

Even though the Carbon MRI is the world's lightest rollator and therefore is easier to lift, it is equipped with a curb lifter that makes it easier to get over smaller obstacles like curbs and low steps. This is particularly helpful if heavy objects are transported in a front-mounted basket or bag.

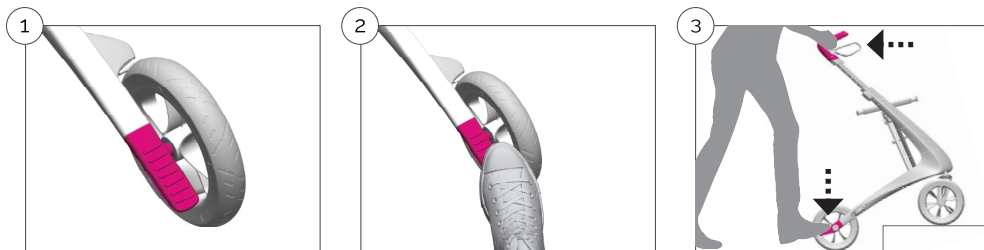- 1-3.** Carefully press the curb lifter with your foot. Pull gently on the handles to lift the front wheels off the ground.

⚠ Be aware that the stability of Carbon MRI changes when it is reclined. Be cautious and try to bypass rather than climb high obstacles.

Intended purpose

The Carbon MRI is designed and tested for people with reduced mobility. It can be used both indoors and outdoors, and it includes a built-in seat for resting when needed.

Accessories

The Carbon MRI can be made even more useful with original accessories. A full display of our range can be found at www.byacre.com.

All of our accessories are model specific and of course tested with it to ensure your safety.

The accessories should always be installed according to the accessories manual. Maximum carrying capacity for bags are always specified in the manual.

⚠ Never use non-original accessories with the rollator, as this could pose a potential risk for your safety.

Maintenance & cleaning

Carbon MRI should be regularly maintained and cleaned to ensure best performance and safe use of the rollator.

Pay special attention to keeping the brakes and wheel hub clean.

All surfaces can be cleaned with a textile cloth and clean water, if necessary, with a little non-abrasive detergent. Wipe with a dry cloth afterwards. Do not use paper towels as these can scratch the surfaces.

Check the screws and nuts on a regular basis. It may be necessary to occasionally tighten one or more screws with a T25 or a H3 key. When adjusting the screws, remember to use Loctite.

Parts subject to wear such as wheels and brakes should be replaced in good time before they may pose a safety risk. Any defective part must be replaced immediately.

We recommend that the seat is regularly impregnated. The seat can be cleaned with a wet cloth and, if necessary, with a little stain remover.

Professional disinfection, as may be required in care facilities, must only be carried out by authorized specialist personnel.

Safety precautions before use

- ⚠ Never use the Carbon MRI if it is in any way defective. Contact your dealer for servicing. In case of any serious injuries related to the rollator, please contact byACRE and/or your national competent authority.
- ⚠ Attention: When unfolding the rollator, please check that the rollator is securely locked in place with an audible click.
- ⚠ Before each use, check that the brakes are working properly and the screws and nuts are tight.
- ⚠ The rollator must not be used by persons weighing over 130 kg/ 286 lbs. It is not suitable for transporting heavy loads.
- ⚠ The rollator is not a toy.
- ⚠ Check that the height adjustment of the handles are correctly engaged and that the handle bars are not set higher than the marked maximum setting.
- ⚠ Do not change/modify the rollator. Such action could negatively impact the safety of using the rollator and expire the manufacturer's liability.

Safety instructions for use

- ⚠ Attention: Always activate both parking brakes before sitting down and while seated. Do not use the rollator to drive a person around and do not move the rollator while seated. The rollator is not designed to carry a person while in motion.
- ⚠ For your own safety, you should not stand on the Carbon Ultralight MRI, nor lean back, sideways or forward while seated.
- ⚠ Do not tilt the rollator more than max. 5 degrees to the side. This could cause risk of falling!
- ⚠ Do not attach bags or other items to the handles. This can cause the rollator to fall.
- ⚠ Always use the brakes when stopping or parking. The brakes can be pulled up a little to decrease the speed while walking.
- ⚠ For good support and safe walking, the rollator must be held close to the body.
- ⚠ Walking upright and looking ahead improves balance. Particular care should be taken when going down curbs.
- ⚠ Do not push the rollator far in front of you as this creates a risk of falling.
- ⚠ Note that the rolling speed of the rollator can be faster than your walking speed and create a risk of falling.
- ⚠ Do not use the rollator on stairs.
- ⚠ Always make sure that your clothes are not trapped or stuck in the handles or in other parts of the rollator.
- ⚠ While riding on public transport, sitting on the rollator is not safe.

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