

BIDET FOR RAISED TOILET SEAT

INTENDED USAGE:

The *KING-BDT-00 Bidet* is a complementary accessory for the KING RTS of height 5, 10 and 15 cm, without soft cushion and with or without lid. It works as a bowl to be used as a container for water and soap for a more accurate and complete personal hygiene after the daily use of the toilet. The use of this accessory allow the user to reach the cleaning level desired without the need of standing from the toilet bowl, reducing slipping risks. The bidet can be applied on both the front or the back of the RTS, for an easy access depending on user's needs.

INDICATIONS AND CONTRAINDICATIONS:

The KING Raised Toilet Seat is aimed to elderly people, wheelchair users, people with reduced mobility of lumbar, knee or hip and to all those might have difficulty in sitting down and getting up from the toilet bowl. The ability to sit independently is required during use. It is not recommended to use the RTS in case of perception disorders or balance problems while sitting.

SAFETY WARNINGS AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the RTS is in a stable position on the WC and correctly fixed on it by the special fixing accessories supplied.
- During the assembly, strictly adhere what is shown in the User Manual.
- Do not modify the RTS and its components in any way.
- Do not use the RTS if damaged or altered in any way. Cracks and deformations may strongly affect the performance and safety of the device.
- Be sure the RTS height is such as to make the use of it easy and safe for the user.
- During use, avoid positions that keep the body weight too much frontward. Wrong positions may lead to instability and overturning of the RTS.
- During the use of the bidet, some water may fall on the floor with consequent slipping risk.
- The RTS and the bidet must not be stored outdoors.

MAINTENANCE INFO:

The products maintenance consists of the cleaning and sanitation procedures. In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or non aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth

Alternatively, the RTS and the bidet can be cleaned in domestic dishwashers at the maximum temperature.

REUSE REQUIREMENTS:


The product is suitable for reuse by other users. Before reuse by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the normal household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose it.


INCIDENT REPORTING:

If there was a serious incident involving this device, contact the manufacturer of this product at info@lmglobaldesign.com and the responsible authority in your country.

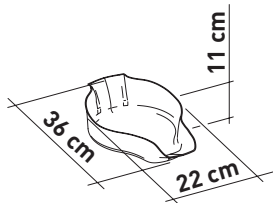


BIDET FOR RAISED TOILET SEAT

1

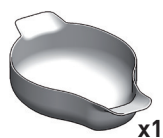

2

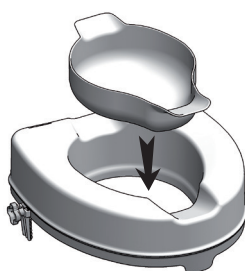

3

**MADE IN ITALY****2 YEARS
GUARANTEE**