

SOFT CUSHION FOR RAISED TOILET SEAT LM

INTENDED USAGE:

The *Soft Cushion for Raised Toilet Seat LM* is a complementary accessory for the RTS of height 5 cm, 10 cm and 15 cm, with and without lid. It makes the RTS more comfortable, making the seat soft and cosy. The soft cushion wraps the RTS, partially protecting it from dirt and making it easier to keep it clean. In addition, the wrinkled surface provides more friction, reducing the slip risk for the user. The soft cushion can be easily applied and removed from the RTS.

INDICATIONS AND CONTRAINDICATIONS:

The Raised Toilet Seats LM are aimed to elderly people, wheelchair users, people with reduced mobility of lumbar, knee or hip and to all those might have difficulty in sitting down and getting up from the toilet bowl. During sitting the tender material of the soft cushion deforms, ensuring an increase of the user support surface and allowing to better distribute the body weight. This makes this accessory particularly suitable if the user suffers from bedsores or diseases which could cause it, such as diabetes. The ability to sit independently is required during use. It is not recommended to use the RTS in case of perception disorders or balance problems while sitting.

SAFETY WARNINGS AND INDICATIONS:

- The maximum user weight indicated in the picture below must not be exceeded.
- Before every use, be sure the RTS is in a stable position on the WC and correctly fixed on it by the special fixing accessories supplied. Also, be sure the soft cushion wraps correctly the RTS.
- Before applying the soft cushion, be sure the RTS is clean and dry.
- During the assembly, strictly adhere what is shown in the User Manual.
- Do not modify the RTS and its components in any way.
- Do not use the RTS if damaged or altered in any way. Cracks and deformations may strongly affect the performance and safety of the device.
- Be sure the RTS height is such as to make the use of it easy and safe for the user.
- During use, avoid positions that keep the body weight too much frontward. Wrong positions may lead to instability and overturning of the RTS.
- The RTS and the soft cushion must not be stored outdoors.

MAINTENANCE INFO:

The product maintenance consists of the cleaning and sanitation procedures. In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or non aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

The soft cushion must be cleaned strictly by hand.

REUSE REQUIREMENTS:



The product is suitable for reuse by other users. Before reuse by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:


Do not dispose of the product in the normal household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose it.

INCIDENT REPORTING:

If there was a serious incident involving this device, contact the manufacturer of this product at info@lmglobaldesign.com and the responsible authority in your country.

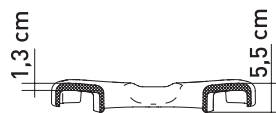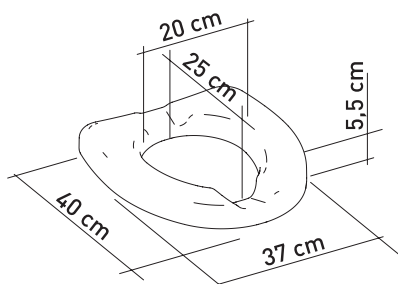

SOFT CUSHION FOR RAISED TOILET SEAT LM

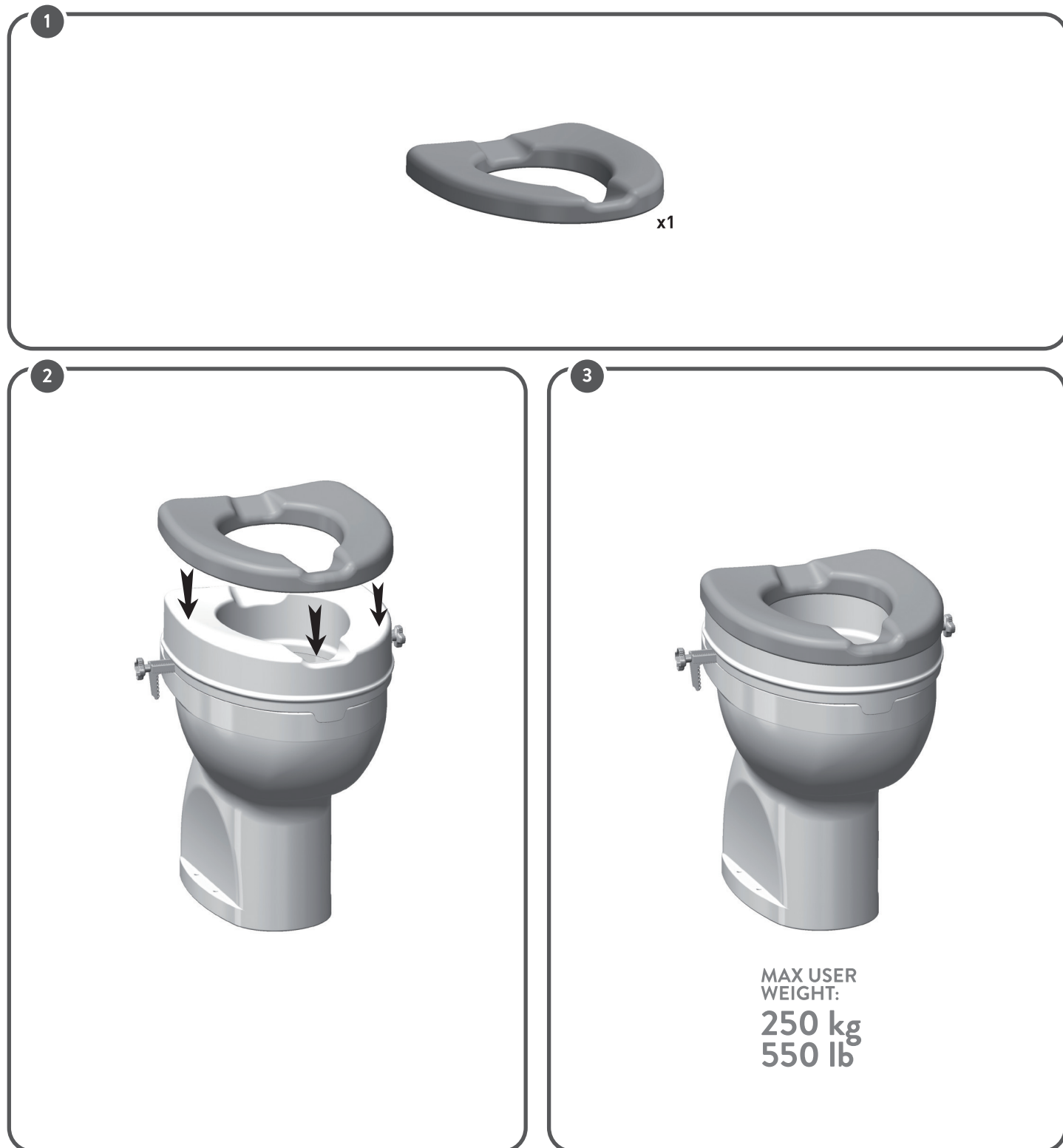

MADE IN ITALY



2 YEARS
GUARANTEE

