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

VARIANTEN

D15.725.000 CULTA LED S 4 B S9
D15.726.000 CULTA LED S 4 B S9 Dewert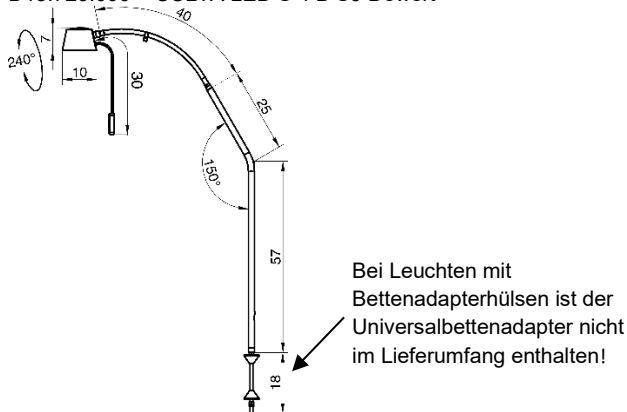

D15.727.000 CULTA LED S 4 P S3

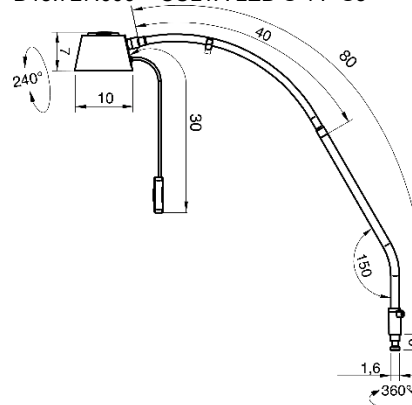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
© Derungs Licht AG • D80.076.000 • 04.2026 • Index: 1.8


Installation- and Operating Instructions

Variations

D15.725.000 CULTA LED S 4 B S9
D15.726.000 CULTA LED S 4 B S9 Dewert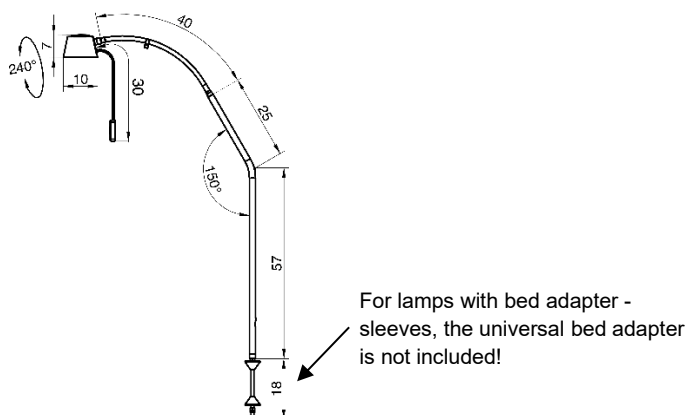

D15.727.000 CULTA LED S 4 P S3

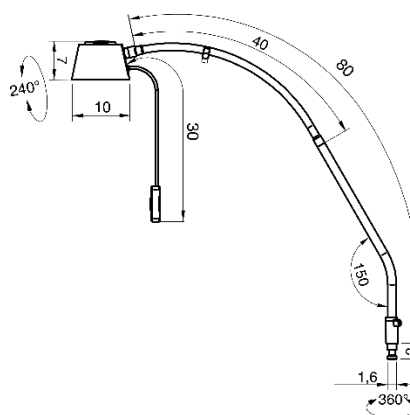1 SAFETY INSTRUCTIONS

1.1 Intended use

Reading and maintenance lamp for use in clinical areas in hospitals and health care buildings where medical treatment, examination and medical care are performed.

1.3 Operating Mode

The luminaire is designed for continuous operation.

1.4 Location

For use in dry rooms only, where there is no risk of explosion. The minimum distance between the lamp head and the illuminated object must be more than 20 cm.


1.5 Safety

- ▶ Do not pull the mains adapter out of the socket by the cable.
 - ▶ The user must comply with the requirements of the national competent bodies for hygiene and disinfection.
- Risk: transmission of diseases.**


2 INSTALLATION

Mount Universal Bed Adapter

- ▶ Connect the supplied bed adapter with the linkage
- ▶ The illustrations below are for models with a universal bed adapter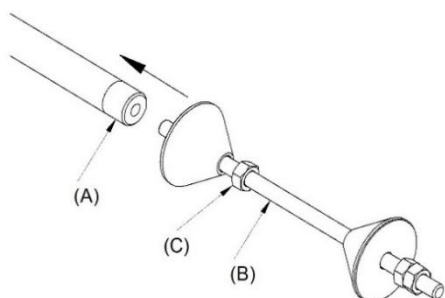

Screw threaded rod **B** into light arm **A** and counter with nut **C**.

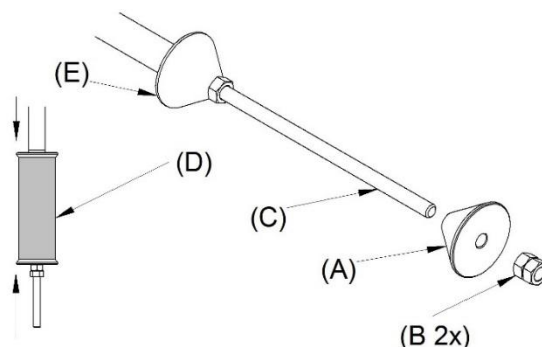- ▶ Remove nuts **B** and taper **A** from the threaded rod **C**.
- ▶ Luminaire with threaded rod **C** and cone **E** in the bedside.
- ▶ Mount cone **A** and nuts **B** and align the luminaire
- ▶ Tighten nuts **B**.

3 COMISSIONING

When the mains adapter is supplied, the lamp must be connected to it. Before commissioning, check whether the connection voltage of the socket agrees with the nominal voltage stated on the rating plate.

4 OPERATION

The light is switched on and off at the cord switch.

5 MAINTENANCE AND REPAIR

DANGER

Before maintenance and repair work, the lamp must be disconnected from the mains! Work may only be carried out by an electrician.

Only parts approved by the manufacturer may be used as spare parts.

6 TROUBLESHOOTING


If the lamp does not light up, first check that the power cord is plugged in and that the outlet is live. If the LED lamp still does not light up, replace it according to the following instructions (Changing the lamp).

7 CHANGING THE LAMP

DANGER

Let the LED lamp cool down

- Make sure the power plug is unplugged and the light is off.
- Loosen screw (2) and remove with protective disc (1).
- Grasp the LED lamp (3) at the edge of the reflector and pull it out carefully.
- Carefully insert the replacement bulb.
- Reinstall protective shield (1) and tighten screw (2) again.

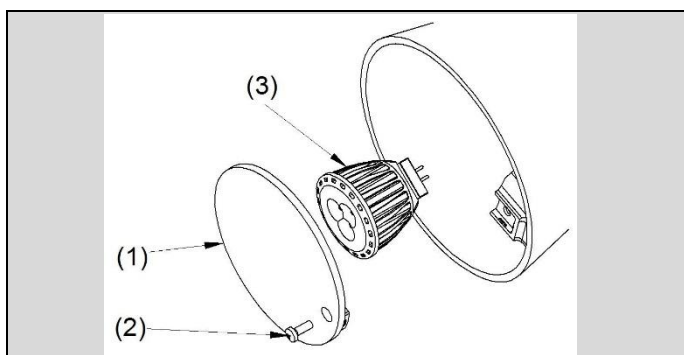Danger

Replace defective LED lamps only with ones of the same type and wattage (see type plate).

8 MAINTENANCE

Regular cleaning of the transparent protective screen guarantees a higher light output. Use a conventional window cleaner for this purpose. To clean the light parts, use a cloth soaked in normal household detergent. Please note compatibility with paints and plastics with the agents used.

ATTENTION

For cleaning, under no circumstances may concentrated disinfectants or solvents, e.g. Sagrotan, Lysoform, ethyl alcohol or other alcoholic liquids be used. Damage by chemical agents, e.g. unsuitable cleaning agents are not the responsibility of the manufacturer.

ATTENTION

Cleaning: The following brands of products may be used to clean the exterior parts of the luminaire:

- ▶ Bacillol AF
- ▶ Descogen Oxy Wipes
- ▶ Descosept Sensitive Wipes
- ▶ Hexaquart XL
- ▶ Incidin Alcohol Wipe
- ▶ Incidin OxyWipe NG
- ▶ Lysoformin
- ▶ Terralin liquid
- ▶ Ultrasol active

9 DISPOSAL

Do not dispose of the lamp and the bulb in the trash. Dispose of the lamp and the illuminant in accordance with local regulations in a disposal center or give it to a dealer with appropriate service.

Cut the cable directly at the housing.

The products listed above are over 95% recyclable. So that after the end of the life of these products, the materials used can be reused materially or energetically to a high extent, the lights are designed to be recyclable. They do not contain any hazardous substances.

10 TECHNICAL SPECIFICATIONS

Electrical Values :

Nominal voltage	100 - 240 V
Frequency range	50/60 Hz
Consumption	4.0 W
Equipping	GL-MR11-HP4W-WW, 30° (Derungs Artikelnummer: D69009000)
Operating Device	Power Supply Unit

Classification :

Protection class	II
Protection	IP20
Operating mode	Continuous Operation
Norms :	EN 60598-1 ; EN 60598-2-25

This product contains a light source in the energy efficiency class:

G

Derungs Licht AG
Hofmattstrasse 12
CH-9200 Gossau SG
Switzerland
Telephone +41 71 388 11 66
Fax +41 388 11 77

Derungs Medical GmbH
Rudolf-Diesel-Strasse 2
78239 Rielasingen-Worblingen
Germany
Telephone +49 (0) 7731
909719-0
Mail:
info@derungsmedical.com

Further distribution partners you find at:

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These mounting and operating instructions are for customer information only and will only be updated or replaced at the customer's request.

Subject to change without notice - printed in Switzerland

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